Instructions for use

Hypersonic Phaco Accessories Set, sterile

H.P. Braem AG
Industriestrasse 4
9552 Bronschhofen
Schweiz

T: +41 (0)71 866 19 33
F: +41 (0)71 866 19 80
E: info@hpbraem.ch
W: www.hpbraem.ch

Precision is our passion

Symboles

Manufacturer

Article
number

Lot
number

Expiry
date

EO
sterilized

Do not use
if damaged

Single use
only

Store away
from sunlight

Keep
dry


Attention


CE Mark
Notified Body

Set content

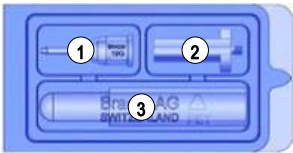Application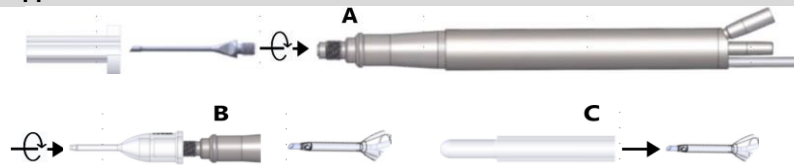

DE-Gebrauchsanweisung

REF12306, REF12307, REF12308, REF12309, REF12311, REF12312

Hypersonic Phaco Accessories Set, steril

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Hypersonic Phaco Accessories Set, sterile

Intended use:

Phaco-Accessories Sets are intended for phacoemulsification for cataract surgery.

Tips, sleeves and sets from H.P. BRAEM AG are tested and approved for use on handles from H.P. BRAEM AG, Oertli®, Geuder®, D.O.R.C.®, Bausch+Lomb® and Hoya-Ruck®.

Contraindications:

There are no specific contraindications for the phacoemulsification. The responsible medical personnel must carefully assess whether the patient is suitable for phacoemulsification.

Risks:

- Damaged instruments or accessories (e.g. damaged sterile packaging, bent or defective tips) must not be used.
- Never touch another instrument with the ultrasonic tip during activation. There is a risk of tip breakage or detachment of parts.
- Capsular Rupture / Infection / CME Cystoid Macular Edema / Increased Intraocular Pressure / Loss of Lens Fragments in Vitreous / Retinal Tear Detachment / Dry Eye Syndrome / Iris Damage

Set contains:

- ① Silikon Sleeve
- ② Phaco tip in key
- ③ Testchamber

Application:

A: Tighten the ultrasonic tip with the key. Remove the key (Attention, do not dispose of the key until the phaco tip has been removed from the handpiece with the key after surgery).

B: Screw sleeve over phaco tip on the handpiece.

C: Fill test chamber with sterile solution and place on phaco handpiece with mounted tip and sleeve. **Perform test run.**

The instructions for use of the Hypersonic Phaco Accessories Set, sterile do not replace the instructions for use of the **phaco handpiece and control unit** used. **Read these instructions for use** and, in case of doubt, contact H.P. BRAEM AG or your sales partner. Observe the respective instructions for use.

May only be used by a specialist! Patient selection and choice of surgical methods are the sole responsibility of the ophthalmic surgeon.

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