Instruction for use

1 step vitrectomy cannula system vitreous hernia

23G sterile

H.P. Braem AG
Industriestrasse 4
9552 Bronschhofen
Schweiz

T: +41 (0)71 866 19 33
F: +41 (0)71 866 19 80
E: info@hpbraem.ch
W: www.hpbraem.ch

Precision is our passion

Symboles

Manufacturer

Article
number

Lot
number

Expiry
date

EO
sterilized

Do not use
if damaged

Single use
only

Store away
from sunlight

Keep
dry


Attention

CE Mark
Notified Body

Application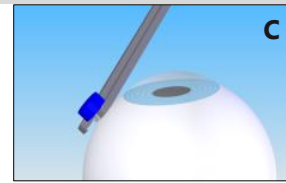

DE-Gebrauchsanweisung

REF 15555

1 Step Vitrektomie Kanülsystem vitreous hernia 23G steril

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REF 15555

1 step vitrectomy cannula system vitreous hernia 23G sterile

Intended use: Trocarsystem for vitrectomy in case of vitreous hernia

Contraindications: There are no specific contraindications for the VCS.

Risks: Hypotony / Wound leakage, potentially requiring intraoperative suturing / Minor subconjunctival bleeding / Vitreous incarceration / Vitreous bleeding / Subconjunctival silicone oil / Cataract / Bacterial contamination of the vitreous cavity / potentially leading to endophthalmitis / Retinal or choroidal detachment

Set contains: 1 inserter sharp (lance cut) / 1 instrument cannula with bulb / 1 closure valve

Caption: **A:** Insertion via Pars Plana into posterior chamber

B: Removing the inserter

C: If finished: remove cannula with valve

Application: The universal cannula is to be used preloaded with the sharp stiletto inserter and closure valve. Perform an incision with the preloaded sharp stiletto inserter via pars plana into the eye till the middle of the cannula shaft. Fixate the cannula with a forceps and retract the sharp stiletto inserter. The vitrectomy system is now ready to start surgery. When surgery is completed, the instrument cannula can be removed by a forceps. In case of small subconjunctival haemorrhage from episcleral vessels, use a cotton wool applicator to press it gently to the area of the scleral tunnel.

May only be used by a specialist! Patient selection and choice of surgical methods are the sole responsibility of the ophthalmic surgeon.

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