Instructions for use

2 step vitrectomy cannula system sterile

H.P. Braem AG
Industriestrasse 4
9552 Bronschhofen
Schweiz

T: +41 (0)71 866 19 33
F: +41 (0)71 866 19 80
E: info@hpbraem.ch
W: www.hpbraem.ch

Precision is our passion

Symboles

Manufacturer

Article
number

Lot
number

Expiry
date

EO
sterilized

Do not use
if damaged

Single use
only

Store away
from sunlight

Keep
dry




Attention

CE Mark
Notified Body

Application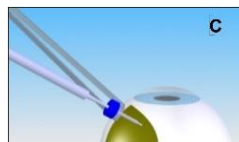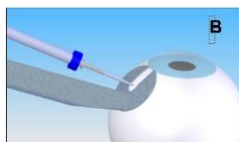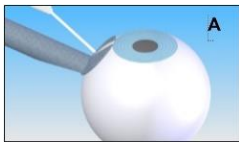

DE-Gebrauchsanweisung

2 step Vitrektomie Kanülsystem steril

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2 step vitrectomy cannula system sterile

Intended use: Trocarsystem for vitrectomy

Contraindications: There are no specific contraindications for the VCS.

Risks: Hypotony / Wound leakage, potentially requiring intraoperative suturing / Minor subconjunctival bleeding / Vitreous incarceration / Vitreous bleeding / Subconjunctival silicone oil / Cataract / Bacterial contamination of the vitreous cavity / potentially leading to endophthalmitis / Retinal or choroidal detachment

Set contains: 3 inserters (trocars) / 3 instrument cannulas with bulb where indicated / 3 closure valves / 1 infusion cannula with infusion tubing and luer / 3 stoppers in a box (only by 23G)

Caption: A: Make an incision* with a micro surgery knife. Distance: 3.5mm. / B: With help of pressure plate insert cannula - insertion angle 30°. / C: While removing the inserter, hold cannula with forceps. / D: Flexible infusion during operation. / E: If finished: removing the cannula with valve.

Application: The 3 universal cannulas are to be used preloaded with an inserter (trocar) and closure valve. Use a pressure plate to move the conjunctiva ± 1 – 2mm sideways and fixate the conjunctiva at this place during the complete insertion procedure. Perform a transconjunctival/scleral incision with a micro surgery knife and insert preloaded inserter at an angle of about 20° – 30° till the middle of the cannula shaft. Thereafter, the pressure plate is no longer needed and can be removed. To overcome the somewhat greater resistance at introduction of the cannula, the inserter must be positioned perpendicular to the sclera to ensure enough counter pressure to prevent the eye from rotating away when pressure is applied. Fixate the cannula with a forceps and retract the inserter. **27G Infusion** - The colored sealing valve must be removed for infusion. Stick the infusion tubing end in the first inserted cannula and switch constant infusion on. Insert the 2 other instrument cannulas in the same way as described above. The vitrectomy system is now ready to start surgery. When surgery is completed, the instrument cannulas can be removed by a forceps and finally cannula with the infusion cannula can be removed. In case of small subconjunctival haemorrhage from episcleral vessels, use a cotton wool applicator to press it gently to the area of the scleral tunnel.

Attention!

May only be used by a specialist! Patient selection and choice of surgical methods are the sole responsibility of the ophthalmic surgeon.

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