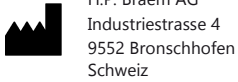
Instructions for use

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EN-Instruction for use

Silicone oil infusion with Braem 23G vitrectomy cannula systems

This document is intended as a supplement only and in no way replaces the instructions for use of the relevant trocar system. Before reading this document, please read the instructions for provided for the relevant trocar set. Only use **original 23G Braem trocar systems** when performing the procedures described in this document!

Intended use: Use of the Braem trocar system for intraocular tamponade with silicone oil up to **5000 cSt**.

Contraindications: **Silicone intraocular lenses can become cloudy as a result of exposure to silicone oil.** Silicone oil should therefore not be used in patients with implanted silicone lenses.

Please also refer to the instructions for use of the oil used as part of the procedure, as well as the instructions for use of the combination ophthalmic devices used (including control devices and instruments). All fluids heavier than silicone oil, such as fluorocarbon liquids, must be removed from the vitreous body before the silicone oil infusion is started.

Maximum infusion pressure on the control module during silicone oil infusion:

50 psi / 3.45 bar

Application:

Attach the cannula in the manner described in the instructions for use IFU_1step/IFU_2step. In order to start an infusion to fill the eye with silicone oil, connect the infusion cannula's luer lock connector to a syringe filled with silicone oil, then connect the syringe to a standard control module. When assembling the device, ensure all connecting surfaces remain free of silicone oil as contamination may result in assembled parts not being securely fitted. You can now start the silicone infusion. Please refer to the relevant instructions for use of the combination devices used.

At the end of the procedure, use forceps to remove the cannulas. Remove the infusion cannula last. Use a cotton tip swab to remove any subconjunctival bleeding.

Attention!

For use by trained specialists only. Selection of the patient as a candidate for a particular procedure is the sole responsibility of the ophthalmic surgeon, as is selection of the surgical procedures involved. / Please refer to the instructions for use.

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